Manufactured by:
SpineGuard S.A.

**Cannulated PediGuard®
Single-use**

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**PediGuard® Canulé
Usage unique**

Assembly Instructions for Cannulated PediGuard®

The Cannulated PediGuard® consists of:

- One Electronic Handle Ref. P2HEXXXX
- One Needle Ref. P2NDXXXX, for example: Needle #1 Ref. P2ND1001

Note: the "X" refers to the different sizes and shapes which are detailed on the product boxes.

The Starter stylets should be threaded completely into the cannula.

The Cannulated PediGuard® Needle must be used with the electronic handle.

Set-up instructions:

1. Inspect the Needle and the Electronic Handle packages for proper integrity.
2. Activate the device by pulling the contact tab out of the Electronic Handle. Activation is confirmed by a beeping sound and green LEDs.
3. Before connecting the Needle to the Electronic Handle, check that the Active Stylet (orange knob) is fully threaded into the Cannula ^[A]. The distal tip of the Active Stylet must be visible at the end of the Cannula.
4. Connect the Needle to the Electronic Handle ^[B]. Ensure snap-on connection of the needle and Electronic Handle is secure.
5. The Cannulated PediGuard® is ready for use ^[C]. Please, refer to the Instruction for Use provided with the Electronic Handle.

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