Manufactured by:  
SpineGuard S.A.

**Cannulated PediGuard®  
Single-use**

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**PediGuard® Canulé  
Usage unique**

#### Assembly Instructions for Cannulated PediGuard®

The Cannulated PediGuard® consists of:

- One Electronic Handle Ref. P2HEXXXX
- One Needle Ref. P2NDXXXX, for example: Needle #1 Ref. P2ND1001

*Note: the "X" refers to the different sizes and shapes which are detailed on the product boxes.*

The Starter stylets should be screwed completely into the cannula.

The Cannulated PediGuard® Needle must be used with the electronic handle.

#### Set-up instructions:

1. Inspect the Needle and the Electronic Handle packages for proper integrity.
2. Activate the device by pulling the contact tab out of the Electronic Handle. Activation is confirmed by a beeping sound and green LEDs.
3. Before connecting the Needle to the Electronic Handle, check that the Active Stylet (orange knob) is fully screwed into the Cannula <sup>[A]</sup>. The distal tip of the Active Stylet must be visible at the end of the Cannula.
4. Connect the Needle to the Electronic Handle <sup>[B]</sup>. Ensure snap-on connection of the needle and Electronic Handle is secure.
5. The Cannulated PediGuard® is ready for use <sup>[C]</sup>. Please, refer to the Instruction for Use provided with the Electronic Handle.

*Note: The Needle XS (ref.P2ND2201) can also be used for posterior sacroiliac fusion as a component of the Cannulated PsiFGuard. The Cannulated PsiFGuard is only cleared in the USA. For the IFU and assembly instruction please refer to <https://www.spineguard.com/products-and-applications/spine-products/user-manuals/>.*

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