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Manufactured by: **Cannulated PediGuard® and C €0123**
SpineGuard S.A. **PsiFGuard**
Single-use US EN

Assembly Instructions for Cannulated PediGuard® and PsiFGuard

The Cannulated PediGuard® consists of:

- One Electronic Handle Ref. P2HEXXXX
- One Needle Ref. P2NDXXXX, for example: Needle #1 Ref. P2ND1001

Note: the "X" refers to the different sizes and shapes which are detailed on the product boxes.

The Cannulated PediGuard® Needle must be used with the electronic handle.

Set-up instructions:

1. Inspect the Needle and the Electronic Handle packages for proper integrity.
2. Activate the device by pulling the contact tab out of the Electronic Handle. Activation is confirmed by a beeping sound and green LEDs.
3. Before connecting the Needle to the Electronic Handle, check that the Active Stylet (orange knob) is fully screwed into the Cannula ^[A]. The distal tip of the Active Stylet must be visible at the end of the Cannula.
4. Connect the Needle to the Electronic Handle ^[B]. Ensure snap-on connection of the needle and Electronic Handle is secure.
5. The Cannulated PediGuard® is ready for use ^[C]. Please, refer to the Instruction for Use provided with the Electronic Handle.

Note: The Needle XS (ref.P2ND2201) can also be used for posterior sacroiliac fusion as a component of the Cannulated PsiFGuard. The Cannulated PsiFGuard is only cleared in the USA. For the IFU and assembly instruction please refer to <https://www.spineguard.com/products-and-applications/spine-products/user-manuals/>.

The Starter stylets should be screwed completely into the cannula.

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
Cannulated PediGuard® and PsiFGuard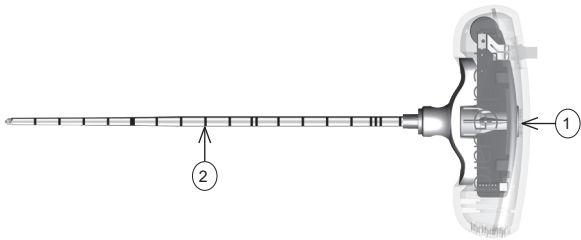

Electronic Handle - P2HEXXXX


Electronic Handle / Poignée électronique / Elektronischen Handgriff / Mango electrónico / Punho eletrônico / Manico elettronico

Cannulated PediGuard® and PsiFGuard Needle - P2NDXXXX

Needle / Aiguille / Nadel / Aguja / Agulha / Ago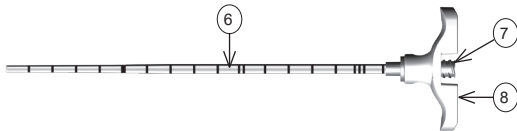


Cannula / Canule / Kanüle / Cánula / Canula / Cannula

Active Stylet / Stylet Actif / Aktives Stilett / Estilete Activo / Estilete Activo / Stiletto Attivo

Cannulated PediGuard® Starter Stylet - P2ST1XX0
Only compatible with Needles P2ND1001 and P2ND1002

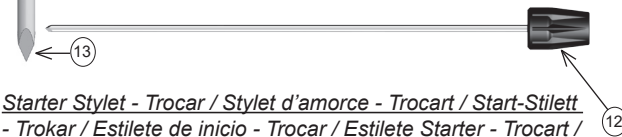Starter Stylet - Trocar / Stylet d'amorce - Trocart / Start-Stilett - Trokar / Estilete de inicio - Trocar / Estilete Starter - Trocart / Stiletto - Trocar a punta di diamante

- | | |
|--|---|
| <p>(1) Electronic handle
Poignée électronique
Elektronischen handgriff
Mango electrónico
Punho eletrônico
Manico elettronico</p> <p>(2) Cannulated PediGuard and PsiFGuard Needle
Aiguille du PediGuard et PsiFGuard
Canulé
Nadel des kanülierten PediGuard und PsiFGuard
Aguja del PediGuard y PsiFGuard
Canulado
Agulha do PediGuard e PsiFGuard
Canulado
Ago PediGuard e PsiFGuard
Canulato</p> <p>(3) Contact Film
Languette d'activation
Kontaktfolie
Lengüeta de activación
Lingueta de ativação
Linguetta di attivazione</p> <p>(4) LED
Diode
LED
LED
LED
LED</p> <p>(5) Electronics (printed circuit board)
Carte électronique (circuit imprimé)
Elektronik (bedruckte Platine)
Circuito electrónico
Sistema eletrônico
Circuito elettrico</p> <p>(6) Metallic Tube
Tube métallique
Metallhülse
Tubo metálico
Tubo metálico
Tubo metallico</p> <p>(7) Luer Lock Thread
Filetage Luer Lock
Luer-Lock-Gewinde
Rosca Luer Lock
Rosca Luer Lock
Filettatura LuerLock</p> | <p>(8) White Plastic T-Head
Embout en T blanc
Weißer Plastik-T-Griff
Cabeza en T de plástico blanco
Cabeça em T de plástico branco
Testa a T in plastica bianca</p> <p>(9) Sensor
Capteur
Sensor
Sensor
Sensor
Sensore</p> <p>(10) Metallic Shaft
Tige métallique
Metallschaft
Eje metálico
Eixo metálico
Stelo metallico</p> <p>(11) Orange Plastic Knob
Bouchon orange
Plastikknopf (Orange)
Pomo plástico naranja
Botão plástico laranja
Manopola in plastica arancione</p> <p>(12) Black Plastic Knob
Bouchon noir
Plastikknopf (Schwarz)
Pomo plástico negro
Botão plástico preto
Manopola in plastica nero</p> <p>(13) Trocar tip
Pointe trocar
Trokarspitze
Punta trocar
Ponta Trocart
Trocar a punta di diamante</p> <p>(14) Bevel tip
Pointe biseau
Schrägschliff-Spitze
Punta biselada
Ponta Bisel
Trocar a becco di flauto</p> |
|--|---|

Important information: Any serious incident relating to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established. For such a report, or for any other problem or malfunction, please contact: Complaints@spineguard.com

[A] Correct screwing of the Active Stylet into the Cannula		Correct distal tip positioning of the Active Stylet into the Cannula	
[A] Vissage correct du stylet actif dans la canule		Positionnement correct de la pointe distale du stylet actif dans la canule	
[A] Korrektes Einschrauben des aktiven Stilett in die Kanüle		Korrekte Lage der Spitze des aktiven Stilett am Ende der Kanüle	
[A] Roscado correcto del estilete activo dentro de la cánula		Posicionamiento correcto de la punta del estilete activo dentro de la cánula	
[A] Roscado correto do estilete Activo dentro da Canula		Posicionamento correto da ponta do Estilete Activo dentro da Canula	
[A] Corretta connessione dello stiletto attivo alla cannula		Corretto posizionamento della punta distale dello stiletto attivo nella cannula	