Instruction For Use of the DSG Connect App

LP2-A073 - Version F

Accuracy

Real Time

Safety

Educational
Tool

Radiation
Free

DSG+
Dynamic Surgical Guidance

Table of content

<i>EN - User manual</i>	4
<i>FR - Manuel utilisateur</i>	10
<i>DE - Bedienungsanleitung</i>	16
<i>PT - Manual do usuário</i>	22
<i>ES - Manual del usuario</i>	30

In no case can the DSG® Connect App replace the surgeon's experience or knowledge of anatomic structures. It is intended to be used by the operating room staff in a clinical environment. The DSG Connect App visually provides the same information as the DSG Connect device provide audibly. The LED and audio outputs from the DSG Connect device are the primary output and should remain the basis for clinical decision-making. THE DSG CONNECT APP IS NOT INTENDED FOR MEDICAL DECISION MAKING.

1. INTRODUCTION

The DSG Connect App is part of a system involving a DSG Connect device and a tablet. A DSG Connect device is a handheld device providing an LED and audible feedback and equipped with a communication module such as the PediGuard, the Cannulated PediGuard, the PediGuard Threaded, the Cannulated PsiGuard and other DSG Connect branded devices from the SpineGuard product portfolio. The DSG Connect App is used to collect, display and potentially record the signal provided by the DSG Connect device, representative of the electrical conductivity measured at the tip of the instrument.

The data transmission with the DSG Connect devices occurs via radio frequency (RF) communication (2.40 to 2.48 GHz, GFSK modulation and maximum RF output power of 9.9 dBm).


2. DSG CONNECT APP TECHNICAL DESCRIPTION

The DSG Connect App is an optional accessory of all DSG Connect devices. It allows the surgeon to collect, display and potentially record the electrical conductivity data measured at the tip of the instrument in real time. The conductivity data are graphically represented by a curve.

The interpretation of the signal should be made as follows: the higher the signal, the higher the electrical conductivity at the tip of the instrument and the lower the signal the lower the conductivity.



Dense tissue, such as cortical bone tissue, have a low electrical conductivity. Cancellous bone has a slightly, but distinctly, higher electrical conductivity. And sacroiliac joint tissue, blood or other bodily fluids have a very high electrical conductivity.

Based upon the different electrical conductivity properties in the different types of tissue, a change in the conductivity signal indicates a change in the type of tissue around the tip of the DSG Connect device.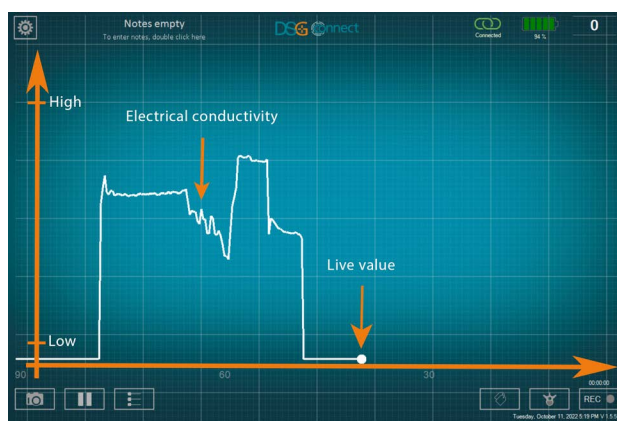

Additional functions are available within the DSG Connect App:

- Screenshot: Allows the user to take a screenshot of the main screen at any specific moment of the surgery.

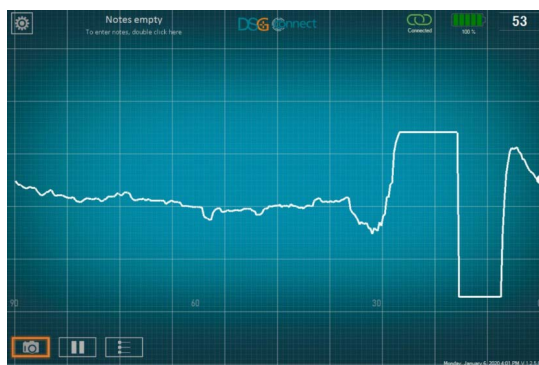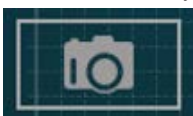- Pause: Allows the user to freeze the main screen and stop the horizontal scrolling of the curve. To activate this function, click on the pause button. A notification will be displayed as a reminder that the live electrical conductivity is not visible. Click once again to resume and go back to the live display.


USER MANUAL (EN)


- Reference lines: Allow the user to manually position surgeon-defined visual lines as reference points on the graph.

- Data recording (available in advanced mode only)
 - Anatomy selection: this feature allows the identification of the anatomical part to be drilled for each drilling record. The drilling position (level and left/right) has to be selected to be able to use the record mode. The data are recorded by individual drillings to facilitate the review at the end of the surgery.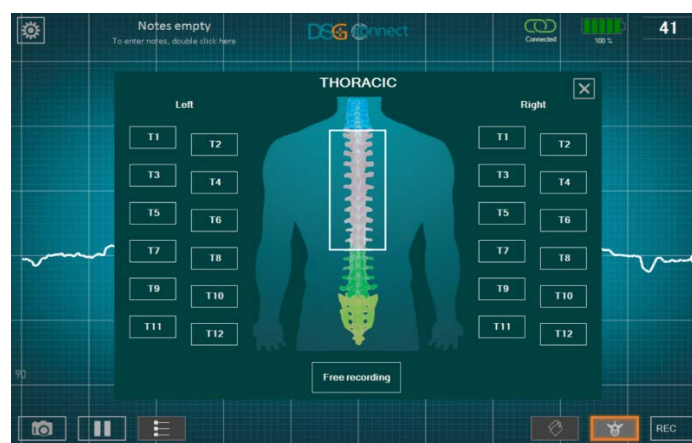

- Start and stop recording: After having selected the anatomical part to be drilled, the recording feature is ready. To start recording, click on the REC button. The curve display resets and scrolls from the left side of the screen, the rec point blinks and turns red, and the time-indicator indicates the duration of the recording. Click once again on the button to stop recording.

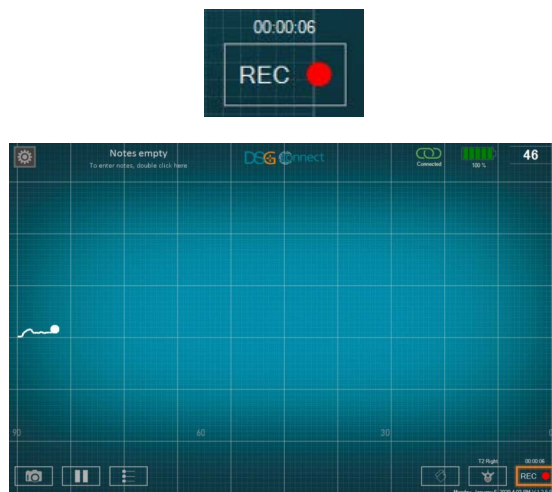
- Tag / Label Positioning: Allows the user to identify a specific event during the surgery by putting a tag on the graph. It is only possible to add a tag during a recording.

The DSG Connect App continuously monitors the battery level of the tablet, the memory level of the tablet, and the state of the connection with the DSG Connect device. If the tablet were to lack **battery** or **memory** or if the **connection** with the DSG Connect device were to be lost, visible and audible notifications would be provided via the DSG Connect App.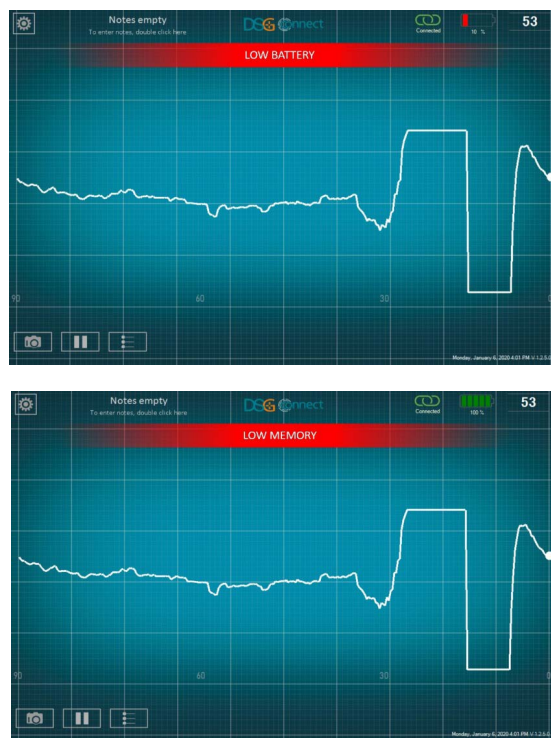

The DSG Connect App detailed settings and functionalities are presented in the Tutorial available in the Parameters menu.

3. BEFORE USE

The tablet must be fully charged before use.

It is recommended to check the available tablet data storage space before use and to clean the tablet memory regularly to prevent the tablet to run out of memory, which could result in the DSG Connect App to slow down or freeze and in the impossibility to record additional data.

The tablet is not sterile and not sterilizable. It is the responsibility of the user to take all the necessary precautions to position the tablet in the operating room.

4. FOR USE

4.1 Positioning of the tablet in the operating room

The tablet should be positioned in the operating room with the screen oriented towards the surgeon for proper visualization.

The distance between the tablet and the DSG Connect device should not exceed 5 meters. The positioning of the tablet should prevent contamination or fall of the tablet on the patient or the operating room staff.

4.2 Pairing instructions

To pair the DSG Connect device with the DSG Connect App:

1. Turn on the tablet, log in the user session "SpineGuard" (no password required) then launch the DSG Connect App.
2. Turn on the DSG Connect device by removing the contact tab.
3. Pairing is automatic. Note: If the automatic pairing does not work, go to the menu Parameters of the DSG Connect App -> Connectivity and click on the DSG Connect device to pair.
4. To confirm the proper connection, perform the preliminary check of the DSG Connect device in a saline solution following the device IFU. The displayed signal should coincide with the LED and audio outputs of the DSG Connect devices. The product reference, lot number and serial number of the connected device could also be checked in the settings menu and compared to the information etched directly on the DSG Connect device.

If the signal displayed on the DSG Connect App does not coincide with the audio signal of the DSG Connect device, do not use the DSG Connect App with this device. The audio and LED outputs of the DSG Connect device should remain the feedback to assist the surgeon in the decision-making process during surgery.

Once a DSG Connect device is paired with the DSG Connect App, it can be manually replaced by another DSG Connect device in the connectivity panel.

4.3 Multiple devices handling

Multiple devices may be active in the room, however only a single device can be connected at a time to the DSG Connect App and the data display will be limited to a single device.

In case the user uses two different DSG Connect devices in a given surgical procedure, "device A" and "device B", the devices connection management to the DSG Connect App should be handled as follows:

1. Follow the pairing instructions as described in section 4.2 for device A.
2. When desired, turn on device B by removing the contact tab. Wipe the tip of the device A to silence it.
3. Go to the menu Parameters of the DSG Connect App Connectivity and click on the Device B to pair. User can use the information displayed in the menu and etched on the device to check the ID of the device (lot and serial number).
4. To confirm the proper connection, perform the preliminary check of the device B in a saline solution following the device IFU. The displayed signal should coincide with the LED and audio outputs of the DSG Connect device.

5. AFTER USE

Once the surgery is completed, exit the DSG Connect app and turn off the tablet.

The tablet is reusable, do not throw it away.

Screenshots and DSG signal data sets recorded during surgery can be accessed after the surgery directly on the tablet via the operating software explorer. All data are stored in a folder named "DSG Connect App data".

6. IMPORTANT MEDICAL INFORMATION

6.1 Warnings

- The LED and audio outputs of the DSG Connect device should remain the feedback to assist the surgeon in the decision-making process during surgery.
- If the DSG Connect App slows down or freezes, do not use the App and check the system.
- Do not use the DSG Connect App if no signal is visualized when the DSG Connect device is dipped in a saline solution or if the signal is not matching the DSG Connect device audio feedback.
- If the DSG Connect App is exited while a recording is in progress or if the connection with the DSG Connect device is lost, data are no longer recorded.
- If the distance between the DSG Connect device and the tablet increases to more than five meters, the connection may be lost.
- The DSG Connect devices are not intended to be connected to any external device and may only connect to approved SpineGuard DSG Connect App and tablet device.
- The DSG Connect App is not intended to be connected to any external device and may only connect to approved SpineGuard DSG Connect device.
- Only use components provided by SpineGuard. Using components from another manufacturer may cause malfunction of the device and be harmful to the patient.
- No modification of this equipment is allowed.
- Do not sterilize the tablet.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the DSG Connect App. Otherwise degradation of the performance of this equipment could result.

6.2 Precautions

- As with any instrument, the tablet should be carefully inspected before use.
- The confidentiality of the data stored on the tablet is the user responsibility.
- The tablet is not sterile, take the necessary precautions for use in the operating room.
- For tablet maintenance instructions, follow the tablet User manual.
- For tablet cleaning instructions, follow the tablet User manual.

7. CYBERSECURITY USER INSTRUCTIONS

7.1 Cybersecurity considerations

The company uses industry-standard instructions to protect the device, including its hardware, system development servers, and the data on those servers by using access enforced firewalls, and SSH secure communications.

The device has been secured using an anti-malware and strong log-in password complexity for configuration access (administrator only).

The device uses radio frequency (RF) communication to receive data. It also has a USB-C port and a proprietary tablet port but these are not intended to receive data. No network communication could be performed with the device (no ethernet port and Wi-Fi disabled).

All recorded data are stored on the tablet memory in the "DSG Connect App Data" directory. All files and folders have a time stamp corresponding to the date of creation.

Only SpineGuard can provide new software version or updates through its distributors or local representatives. User must not attempt to download device update.

If device cybersecurity has been compromised, or is suspected to be, do not use the DSG Connect App. A signal discrepancy between the DSG Connect device and the DSG Connect App could announce anomalous conditions and could allow detection of a potential cybersecurity issue. In any case, the LED and audio outputs from the DSG Connect device are the primary output and should remain the basis for clinical decision-making. THE DSG CONNECT APP IS NOT INTENDED FOR MEDICAL DECISION MAKING.

7.2 Device Security

It is the responsibility of the authorized user to ensure that the device is not left unlocked, or otherwise unsecured when not in use, to ensure that non-authorized medical, professional, or otherwise unapproved personnel are not exposed to, or gain access to, the device.

7.3 User Responsibility

Users must not take snap-shots, screenshots or pictures (e.g. using another device) of any information viewed through the device.

Operators should not enter identifying data into the device. All the data on the device should be de-identified and related to the sample ID and not to the patient. Free fields must not contain personal data and/or clinical data allowing identification of a patient.

7.4 Reporting Device Security or Privacy Breaches

Operators must contact their local IT department to disclose any suspected or confirmed privacy or security breaches.

7.5 Recovering from Compromised Accounts or Devices

When devices are lost, or unauthorized access is discovered or suspected, operators must contact their local IT department or SpineGuard and must not use the device. If corrupted, no attempt must be made by the user to restore features or configuration.

7.6 Unavailable Service

Users should report unavailable service or prohibited access to information to their local Healthcare Organization's IT department.

7.7 Software Bill of Materials (SBOM) and Architectural Diagrams

In order to prevent the public disclosure of potential cybersecurity vulnerabilities, the SBOM and architectural diagrams (including port control) shall only be available to customers upon request; and if agreed upon in the customer services agreement. Please contact support for further information.

7.8 Coordinated Vulnerability Disclosure and Updates

In a case where a vulnerability has been exploited in the device, SpineGuard will follow its internal procedures to investigate, assess the impact and contain the incident and determine if the vulnerability is controlled or uncontrolled.

Notification of the vulnerability, its scope and effect, SpineGuard's resolution and any actions required by the user will be communicated to affected users on a specific schedule based on the risk of the vulnerability, via phone or email and the company website.

SpineGuard intends on notifying customers of uncontrolled vulnerabilities within 30 days of awareness, and providing fixes within 60 days.

7.9 End of Life and Decommissioning

At the end of support, the company will no longer offer access to the device. The user will be notified with time to retrieve any needed data from the system through the coordinated vulnerability disclosure process.

CAUTION: FEDERAL LAW (US) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED PHYSICIAN.
IF YOU HAVE QUESTIONS ABOUT THIS DEVICE, PLEASE CONTACT:

SpineGuard, S.A. (Manufacturer)
10, Cours Louis Lumière
94300 Vincennes France
Phone: +33 (0) 1 45 18 45 19
Fax: +33 (0) 1 45 18 45 20

Symbols found on the product are described below:

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SpineGuard, S.A. (Fabricante)
SpineGuard® S.A.